

Self-donning
safe removal

Self Gown

3 simple donning steps



Slide your hands into the sleeves.




Tear along the perforation and tie the belt.



Donning and removal procedures



One touch to take off

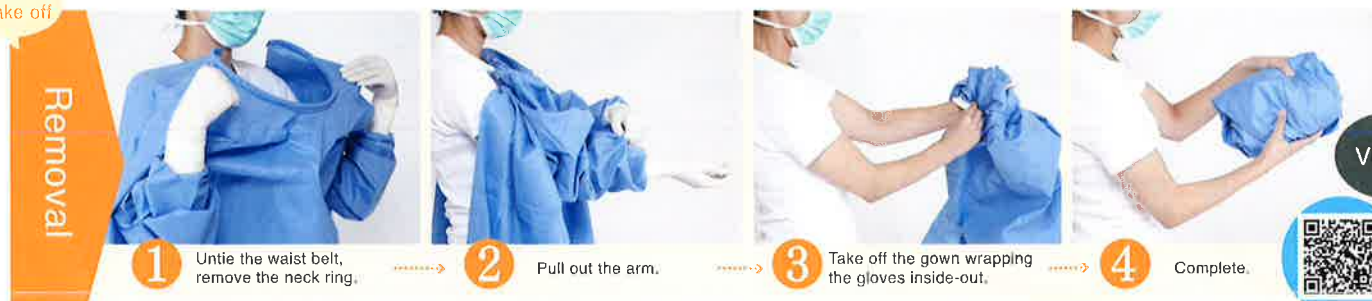

Gown features and benefits


- By comprising a special spring along the neckline, the wearer can don the gown without assistant tie the strings or fasten hook-and-loop in a conventional gown.
- The spring characteristic of the neckring provides the wearer the flexibility to bend over.
- The inner belt is removed by applying larger rear torso portions to overlap each other.
- Applying sticky tape on a transfer card (figure 1) for temporary adhesion and forming a special perforation (figure 2) at the end of the waist belt enable the wearer to self-tie the waist belt maintaining sterility.
- Removing the gloves with the gown will not splash blood and can prevent environmental infection.
- Selfgown can also be donned and removed quickly and safely without an assistant besides in ER for therapeutic procedures at outpatient clinic or CVC at hospital ward.

Prevent onwards transmission to environmental infection from splash.

Selfgown can minimize environmental infection from splashes as the wearer can take off the the gown wrapping the gloves inside-out at the same time, as opposed to a conventional gown where the wearer needs to remove the gloves first in order to remove the gown, which cannot prevent droplet infection.

Splash test using artificial blood.

We tested droplet infection using a conventional gown. As on the right, spattering blood was found when taking off the gloves, as opposed to Selfgown, there was no splash found.

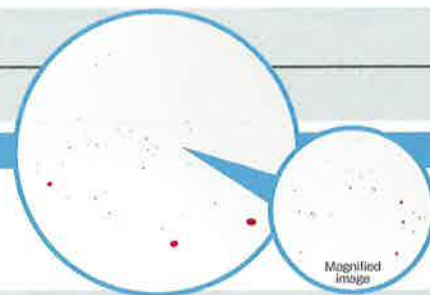

Product code	Item name	Spec	Unit	Quantity		latex	Barcode
				Inner box	Outer carton		
40459	Self gown L (G)	Side knot type,L,individual pack,sterilized	bag	25bags	2boxes/carton	Free	JAN : 4904820404597
40460	Self gown XL (G)	Side knot type,XL,individual pack,sterilized	bag	25bags	2boxes/carton	Free	JAN : 4904820404603

Selfgown is a registered trademark of Daiei Co., Ltd.

Daiei Co., Ltd.



2-3-3 Miyakojima-hondori, Miyakojima-ku, Osaka 534-0021 Japan
 TEL : +81-6-6924-0454 E-mail : info-eng-01@amethyst.co.jp
 FAX : +81-6-6928-9450 U R L : http://www.amethyst.co.jp